

NECK AIR TRACTION DEVICE

NECK PAIN SOLUTION

Life Can Feel Good Again




MANUAL

Please read this manual before using the device

Product introduction

This neck traction device is a patented device that is designed to adjust the user's cervical spine to the natural physiological curve 20°. The adjustable screws in the left and right side can be adjusted up and down to the desired traction strength. The back air-bladder will gently lift the user's cervical vertebra by means of vertical air columns. It is lightweight and easy to use. It may allow performance of daily standing, sitting and office working for extended periods of time.

Components & Accessories

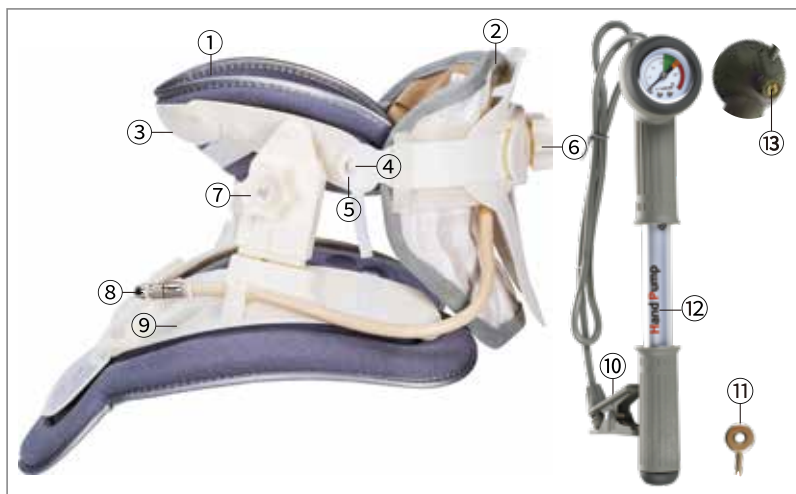


- | | | |
|---------------|---------------|-------------------|
| ①support pads | ②air bladder | ③jaw support |
| ④knob | ⑤pull tab | ⑥back screw |
| ⑦side screw | ⑧fill valve | ⑨shoulder support |
| ⑩air clamp | ⑪valve wrench | ⑫hand pump |
| ⑬relief valve | | |

Operation Instructions



1. Please tighten the valve clockwise before inflation.

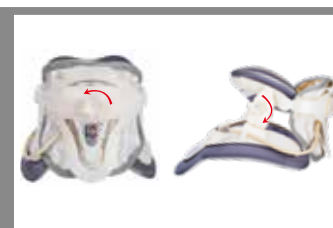

2. Please turn the back screw anti-clockwise to the loosest and both side screws to the bottom.



3. Rise your head slightly, position the front panel directly under the chin.



4. Push the front panel towards your neck to fit and fix the pull tab to the knob.



5. Turn the back screw clockwise to the proper size you may feel comfortable, you can turn it anti-clockwise if it is too tight.



6. Turn the side screws forward to the proper traction strength, the strength is up to individuals. You can turn the screws backward if it is too strong.




7. Attach the clamp from the air pump to the fill valve.



8. Use the hand pump to inflate, to a comfortable stretch.





9. Remove the clamp from the fill valve.

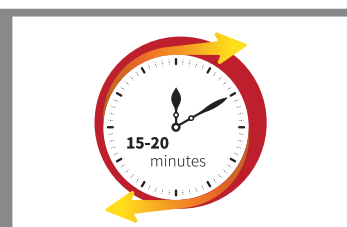


10. Leave it in place for the next 15-20 minutes or for the time recommended by your physician.

11. When the treatment period is complete, release the air-pressure, turn the back screw anti-clockwise to loosest and turn the side screws backward to the bottom. Release the hasp from the knob and take the device off from your neck.

Handpump Instruction

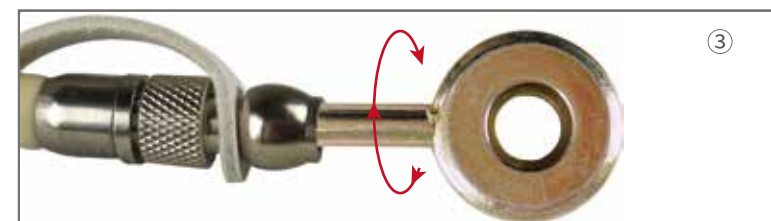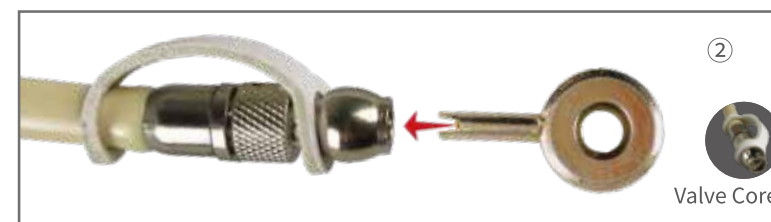

1. Please tighten the valve clockwise before inflation.(As shown on Figure 1)
2. Please tighten the valve core every one month.(As shown on Figure 2&3)
3. When inflating , if there is compression leak or no flowing transfer. Please tighten the valve core according to the Figure 2&3

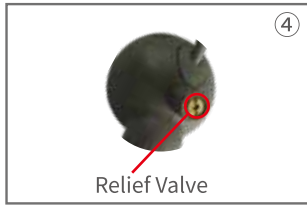

4. When the air bladder is full filled, you will hear the sound of the air leak because the relief valve start working. The pressure relief valve will protect the air bladder from blowing up.

5. After inflation reaching the desired pressure, please remove the clamp immediately.



Indications for use

1. Keep the cervical spine stable.
2. Cervical disc herniation.
3. Immobilize neck according to physician's suggestion after cervical surgery (inflate before wear).
4. It can be used as the substitute for plaster, splint to immobilize neck and head.


Maintenance & Cleaning Instructions

1. With a clean damp cloth ,wipe down the device and air bladder. Do not bleach or submerge in water.
2. Take apart the fabric pads and clean with some mild clean detergent. Do not machine wash.
3. Air dry only. Avoid direct sunlight and heat exposure.
4. Avoid the device contact with any sharp objects
5. Do not wash the support pads too often

Cautions And Warnings

1 Consult with your physician before using this product.	2 Must remove any jewelry, earring, necklace, eyeglasses or any other item that could get in the way of therapy.
3 Wear this device for 15-20 minutes each time, 3-4 times each day.	4 Do not use this device if you are pregnant or under age 12 .
5 Do not use this device if you have a disease malignancy or cancer.	6 Do not use this device if you have spinal cord spondylopathy.
7 Avoid device contact with sharp objects.	8 If you experience or feel any adverse reaction to this product, discontinue use of the product and contact your physician.
2. If you experience dizziness, nausea, undue comfortable or numbness, discontinue use of the product and contact your physician.	2. Please keep regular exercises in conjunction with the use of this device.


Electric Inflator Pump

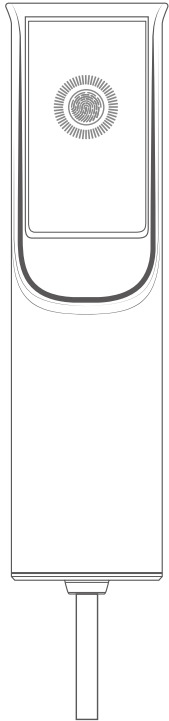


MANUAL

Please read this manual carefully before using.

Inflating Instruction

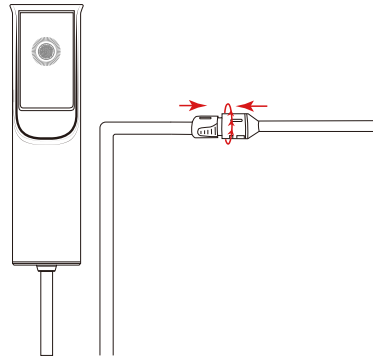


1. Connect the pump to the valve , hold and turn the pump side valve forward to lock.

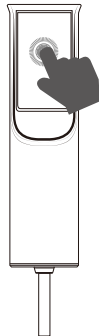

2. Press and hold the “  ” button to inflate until you feel the comfortable traction . Release to stop inflating.

Charging

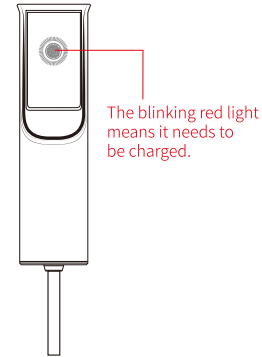

- When the red light blinks, or the electric pump fails to charge the product, it means that the pump needs to be charged.
- The red light will stay on when being charged. When fully charged, the light will turn to blue.

Notes

- Do not keep the electric pump charging the neck traction device for a long time, which may damage the product and the pump.
- The electric pump can be only apply to matched neck traction device, please do not use it on any other products.
- Keep out of the reach of children



- The battery used in this product is a special rechargeable battery and shall not be used in other products.
- Do not charge the rechargeable battery that has been removed.
- Do not throw it into the fire.
- Do not puncture with sharp objects, impact by gravity or disassemble.
- Do not short circuit the positive and negative poles of the battery directly with metal connections.
- The above situations can cause heating, fire, rupture and other phenomena.

Technical parameters

Rated voltage: 3.7V
Current: 1.6A (MAX)
Charging voltage: 5V
Battery capacity: 1200mAh
Flow volume: 2.8-4.5 LPM