

## Charging

1. When charging the hearing aid, it can be charged through the power adapter or through the USB interface
2. Hearing aid charging: first turn off the power of the hearing aid, and then charge it. The first charge should ensure sufficient charging time (not affected by the indicator light), and a charging time of 10–12 hours should be guaranteed
3. If the hearing aid is not used for a long time, the battery self-discharge will cause the battery to be seriously insufficient. It should be charged, and the charging time should reach 12 hours. Even if the charging indicator shows that the charging is complete, the charging should not be stopped (guarantee to reach 12 hours )

11

### Exercise

Listen carefully to a song you are familiar with or watch a TV game show, and guess the answer without looking at the subtitles.

## THE SECOND WEEK

The more you wear the hearing aid, the more you can adapt to the sound environment. You even can ask people around you not to speak loudly to you anymore, because you can easily communicate with people.

15

4. During use, the hearing aid volume becomes low or when the hearing aid is not in use, the power should be turned off and charged (this will better protect the hearing aid battery). This is very important! Otherwise it will cause excessive discharge
5. The hazards of excessive battery discharge will reduce the battery capacity and affect the battery life to varying degrees
6. When using AC power to charge, if the charging is over, unplug the power in time

12

## THE THIRD WEEK

After a few weeks, you will feel more energetic because you no longer need to spend a lot of energy to listen what others say. The sound will become clearer and clearer, and you will once again enjoy the sounds that surround you in your life, such as the melodious singing of birds in the backyard, the cheerful laughter of children, and the rustling of leaves in the wind.

### Participate

Have a meal in a restaurant with friends, watch a movie in a theater, join a family gathering, and no longer have to bother to speculate about missed information.

16

## Volume control



As shown in the figure: Touch the center of the shell to adjust the volume, there are 5 levels of volume, and the volume level is 1--5 circular adjustment;

A. Tap once, the hearing aid will prompt a "beep" and increase the volume by one level. When three beeps are heard, it means that the volume has reached the maximum level (each tap prompts a "beep")

※ When adjusting the volume, please do not cover the sound pickup hole on it

09

## Charging

Charging compartment

Flashing green = charging

Steady green light = charging is complete

Hearing aid

Steady blue light = charging

The blue light is off = charging is complete

Hearing aids can be used for 35 hours at a time

The charging compartment can charge a single hearing aid 4-5 times



Charging



Charging completed

10

## 21-day wear chart

### THE JOURNEY TO LISTENING

Unlike vision correction, which simply wears glasses, hearing correction is a rather detailed process. Once you start your hearing correction journey, you will experience countless "new sounds", and your brain needs to adapt for a period of time. In just a few weeks, you will be shocked by the sound that you have never experienced before.

If you have any questions, we will serve you wholeheartedly.

13

## THE FIRST WEEK


Some sounds may be too noisy when wearing a hearing aid for the first time, such as the buzzing of the refrigerator, the wrinkling of paper, or the sound of your footsteps. This is normal, and your brain is relearning how to prioritize these sounds that you don't hear very often. Sometimes you may not like the noise, but please don't give up wearing.

### Patient

At the beginning, wear the hearing aid for 30 minutes a day, and then gradually increase the wearing time until the whole day.

14

## How to wear

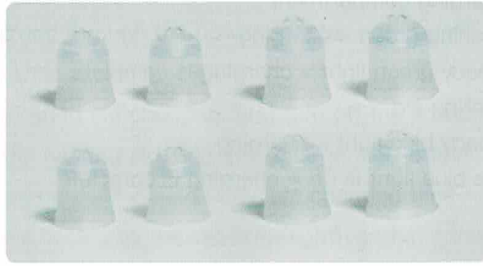


1. Choose earplugs that fit your ear canal size



2. Install the selected earplugs and the machine

03

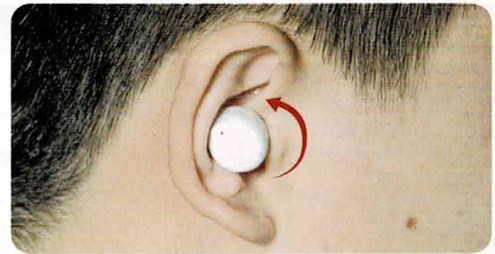

3. Wear the hearing aid as shown in the figure, and rotate the hearing aid to the ear canal as shown by the arrow in the figure



4. Adjust the appropriate volume program

※ When adjusting the volume, do not cover the pickup port

04

## Turn ON

After the hearing aid leaves the charging compartment, it will automatically turn on after 10 seconds, and you will hear a "beep" sound. You will have plenty of time to wear your hearing aid to prevent howling/acoustic echo during the wearing process.



- ※ Please avoid covering the hole when operating the device
- ※ The first charge should be guaranteed to reach 10-12 hours

07

## Turn OFF

After use, put the hearing aid directly back into the charging compartment to automatically shut down. The product does not support manual shutdown



- ※ Low battery prompt: four fast beeps
- ※ The chip of the charging box has an accurate detection function, and the earphone is taken out for 3 seconds and then returned to the warehouse for accurate charging.

08



# OPERATION GUIDE



01

**Note:** Be sure to choose earplugs that fit your ear canal and wear them correctly to fit your ears as much as possible. If you don't wear it properly, there will be echoes (whistles/sounds).

**Note:** After wearing, you can turn on the hearing aid to adjust the volume and program! !

## Problems encountered in wearing


### 1. There is no sound when wearing?

If there is no sound:

- ① Check whether the volume program is adjusted to suit your hearing
- ② Check whether the hearing aid has electricity
- ③ Whether there is no episode in the ear canal, and the earwax is blocking the sound of the hearing aid.

05

## Before wearing

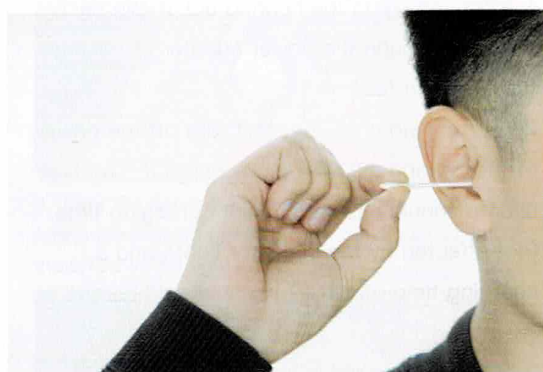

Please use a cotton swab to clean the ear canal before wearing. If the cerumen is too much, it will block the sound outlet of the product and cause no sound.

02

**Note:** If the hearing aid is silent or the sound becomes low due to the blockage of the sound outlet, please clean the sound outlet.

### 2. Why is there a howling (sharp sound)?

Howling (sharp sound) is a manifestation of the normal working of hearing aids. The hearing aid and the ear canal are not completely matched, causing the sound from the receiver to be received by the microphone again, and the sound forms a loop, which will be continuously amplified and produce howling.

**Solution:** The product is equipped with earplugs of different sizes, please choose the earplugs that fit your ear canal

06