

INSTRUCTIONS:

CaneLight has two options:

- 1 - Illuminates path
- 2 - Blinks red SOS Morse Code for emergencies

First push: Enables LED light (do **NOT** look at the light directly.)

Second push: Enables flashing red LED in Morse Code*

Third push: Enables night time locator light (visible in the dark only.)

Fourth Push: turns CaneLight off.

**Flashes red SOS Morse Code indicating an emergency.*

Powerful 3 Watt COB light generating a 50' beam, 6 hours of continuous use.

Lever to place on table surface for easy access.

To recharge:

Plug the supplied USB cable into CaneLight and the other end of USB cable into a USB power source.

Unit will turn green with full charge.

Green = Fully charged.



Red = Battery low, needs recharge.

