



Folding Seat Crutch

SP0590

USA office: Fontana GBR office: Ipswich AUS office: Truganina

If you're having difficulty, our friendly customer team is always here to help.



USA:cs.us@costway.com AUS:cs.au@costway.com GBR:cs.uk@costway.com





Before You Start

- ⚠ Please read all instructions carefully.

- ⚠ Read through each step carefully and follow the proper order.
- ⚠ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⚠ Always place the product on a flat, steady and stable surface.
- ⚠ Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.

Introduction


Dear customers:

- Thank you for using our seat crutch.
- Before using it, please read the instruction manual carefully and use it correctly.
- Keep this manual in a safe place for future reference.

Product Introduction

- Seat crutch is a multipurpose and convenient crutch with a seat, When open, it is a triangular chair. When folded, it becomes a sturdy crutch. The three legs with rubber feet are more stable than other crutches. The main structure is aluminum alloy, with durable ergonomic ABS seat board, ergonomic handles, and the seat crutch is adjustable in height so that users can adjust it according to their height.
- Seat crutches bring the crutches a new definition, bring the leisure chair a turning innovation. The perfect combination of the characteristics of crutches and leisure stool, the most significant increase in the safety of crutches and leisure stool, which is the most trusted leisure chair and walker crutches to family.
- Seat crutches suits for leisure-loving old people, elder who wait in line, can help to release their burden, and also fit for people who like to go fishing, and people who want auxiliay functions.




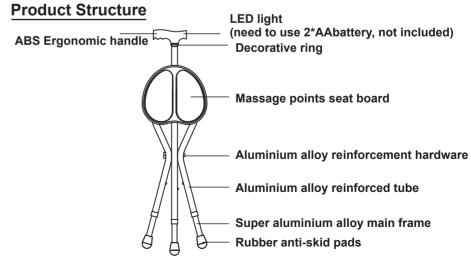

02

Steps for Usage

How to Use When Folded

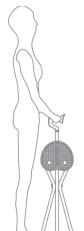


First Method:
Hold the handle forward
This kind of usage can
massage a lot of
acupoints of the hand to
achieve a variety of
beneficial effect to
human body.




Second Method: Hand grip backhand This method can massage Hoku, this one is more comfortable than the method one.

Usage When Open the Seat Crutch


Open the Seat Board

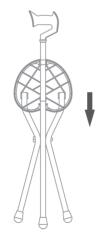

When open, pull downward the seat board to the fixed position. When folding, pull back and remember to fix the seat board to the buckle.

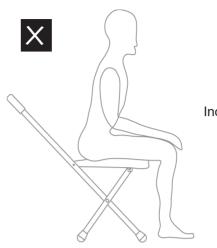

Incorrect Usage

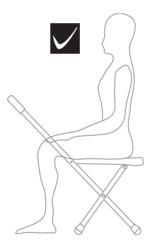

The right usage:

After opening, it is a beautiful heart-shaped seat board. It comes with massage points of which the height is in accordance with the ergonomics. There are two big dots designed to massage Huantiao (GB30) point, to prevent movement series diseases: sciatica, lower limb paralysis, cerebrovascular disease sequelae, lumbago and leg pain, hip and surrounding soft tissue diseases, beriberi, etc.

Precautions

Please note the following:

- 1. The wire on the floor
- 2. The liquid on the floor
- 3. Basy sliding carpet

- 4. The stairs
- 5. The courtyard of the door
- 6. The gap of the floor

04 05





Return / Damage Claim Instructions

⚠ DO NOT discard the box / original packaging.

In case a return is required, the item must be returned in original box. Without this your return will not be accepted.

\triangle Take a photo of the box markings.

A photo of the markings (text) on the side of the box is required in case a part is needed for replacement. This helps our staff identify your product number to ensure you receive the correct parts.

\triangle Take a photo of the damaged part (if applicable).

A photo of the damage is always required to file a claim and get your replacement or refund processed quickly. Please make sure you have the box even if it is damaged.

$\ensuremath{\Delta}$ Send us an email with the images requested.

Email us directly from marketplace where your item was purchased with the attached images and a description of your claim.