

Instructions

- Step 01** Connect the cord of the controller to the motor socket
- Step 02** Connect the adaptor cord to the motor socket which will control the push rod.
- Step 03** Insert the plug into the power supply
- Step 04** Lie on the bed, press the up and down switches on the controller to control the lifting mechanism.



Size Description

Capacity: 150kg-170kg
 Size: L 120cm x 75cm
 Thickness: 4.3cm
 Weight: 12kg
 Back angle: 0-80

Voltage: 100-240V 50/60HZ
 Motor Power: 45W
 Control Cord length: 1.2m
 Power Cord length: 2m



Cautions

- Never use when sleeping with a baby or toddler.
- Use extreme caution when sleeping with a partner
- Use only as attended
- Use extreme caution if using this equipment with other equipment i.e. CPAP or Oxygen.

CRUSH RISK – This is a motorized device that **DOES NOT DETECT RESISTANCE**. This product will attempt to close on any fingers, toes, arms, legs, cats, dogs, and babies behind cushion, without regard for their well-being. Use extreme caution when using this product.


Main Instruction Manual:

- 1 Remove all components from the carton and remove all packaging, and place the backrest body on a bed or level surface.
- 2 Connect the product as shown below, and ensure that the power adapter is not under the backrest frame to avoid crushing when lifting.
- 3 Insert the power plug into the socket. Note: The power plug must always be kept accessible so that it can be immediately removed from the outlet in an emergency.

Operating instructions

The back lift pad can be operated by the manual control panel button. When not in use, the manual control panel should be placed in a safe and observable position to avoid accidental touch, only one button at a time to avoid possible abnormal damage to the system, such as operation does not respond, please first check whether the parts are correctly connected and in a normal insertion state.

Connection Schematic

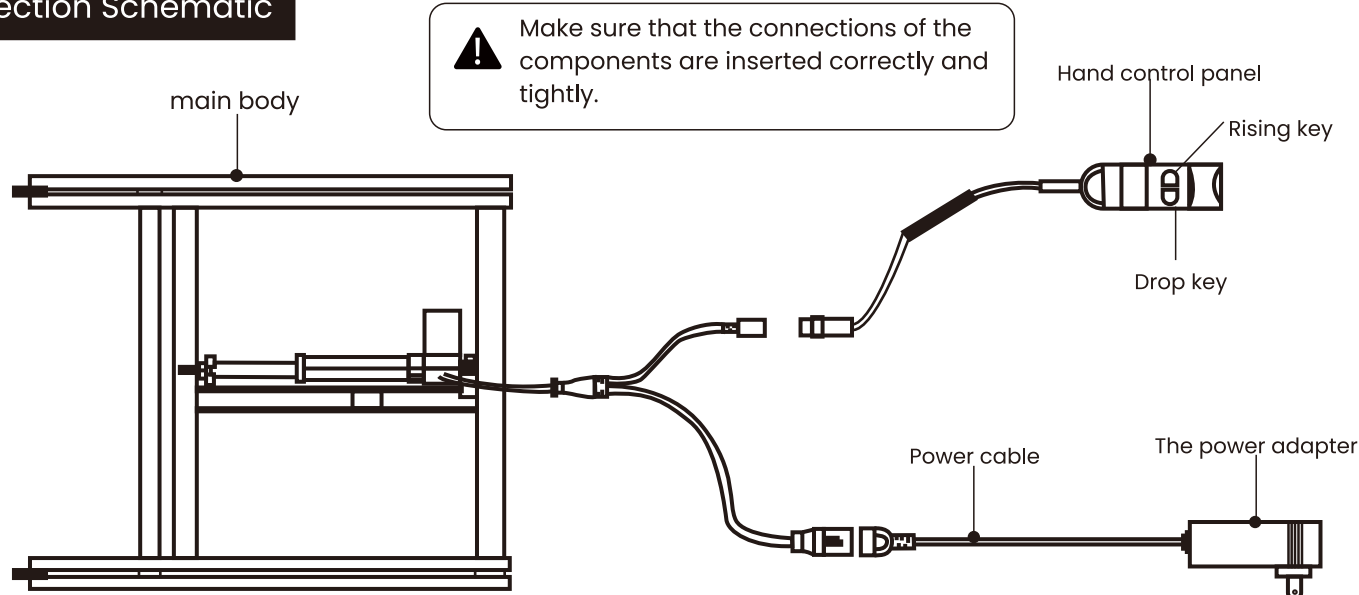

⚠ Matters need attention

- 1 If the operating product does not respond, please re-plug the connections and ensure that they are properly connected and inserted tightly.
- 2 Before installing the product, please ensure that the connecting lines and other components are in the right position to avoid being crushed or torn during movement.

 Technical Data: Safe Workload:150kg

 Backrest adjustable range: 2 ° -80 °

 Applicable voltage: 110V ~ 220V