

1. Sliding Seat User Guide



a. The soft plate should be centered to start



b. Slide the plate to the side over the bathtub edge



c. Sit on the soft plate and rotate so you can put your legs into the bath tub



d. While sitting, slide the soft plate over to the center of the bath. You are now in the proper position for use.

WARNINGS

1. DO NOT use the MobiLift on any carpet, rug or any non-smooth surface. The suction cups require a hard/smooth surface to properly stick to the ground.
2. Unplug the device before removing any parts.
3. Close supervision is required when used around children or pets.
4. DO NOT use the MobiLift for anything but its intended purpose.
5. DO NOT connect to power if the electrical cord or device is damaged in any way.
6. Keep the device away from intense heat sources.
7. Never drop or insert any object into any opening.
8. Make sure the charger and the hand control connector remain dry at all times. The hand control itself can get wet when being used in the bath, however it should not be submerged.
9. Ensure the lift is in the neutral position before unplugging from the outlet.
10. Unplug the MobiLift from the outlet when not in use.
11. Keep all fingers clear of the lifting mechanism when in use.
12. Do not allow direct contact with any type of intense heat source.
13. Consult your physician to ensure there aren't any medical reasons that may prevent safe usage of the MobiLift.
14. Ensure the MobiLift is clean and dry before putting it in a safe storage location.
15. If there is any sudden impact to the back of the lift, it may be necessary to readjust the base to ensure it is properly suctioned to the ground.

DANGER - Always remove the MobiLift from its power source when cleaning to minimize the risk of electric shock.