USA office: Fontana **AUS** office: Truganina **ITA** office: Milano **POL** office: Gdańsk
DEU office: FDS GmbH, Neuer Höltingbaum 36, 22143 Hamburg, Deutschland **FRA** office: Saint Vigor d'Ymonville
GBR office: FDS Corporation Limited, Unit 4, Blackacre Road, Great Blakenham, Ipswich, Ip6 0FL, United Kingdom

If you're having difficulty, our friendly
customer team is always here to help.

USA:cs.us@costway.com
AUS:cs.au@costway.com
GBR:cs.uk@costway.com
ITA:cs.it@costway.com

POL:sklep@costway.com
DEU:cs.de@costway.com
FRA:cs.fr@costway.com
ESP:cs.es@costway.com

Two in One Rollator / 2-in-1 Rollator
Fauteuil Roulant 2-en-1/ Rollator Dos en Uno
Rollator Due in Uno / Chodzik dla seniora 2 w 1

JH10010

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION.
PLEASE READ AND KEEP FOR FUTURE REFERENCE.

EN

Before You Start

- ⚠ Please read all instructions carefully.
- ⚠ Retain instructions for future reference.
- ⚠ Separate and count all parts and hardware.
- ⚠ Read through each step carefully and follow the proper order.
- ⚠ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⚠ Always place the product on a flat, steady and stable surface.
- ⚠ Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.

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● INTRODUCTION

The rollators are manufactured from high-quality materials and include a seat, bag, and locking brakes. The hand-bar are adjustable in height and the rollator folds easily for transport or storage.

● INTENDED USE

Designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown. The intended user should possess the necessary physical, visual, and cognitive skills to operate the rollator safely. If you have any doubt, please consult us or your prescriber. It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

● SAFETY ADVICE

This rollator is designed to be used as a mobility aid, creating a natural gate as the user walks securely with their primary weight between the handle bars with the unit in front of the user. Read this manual carefully before using your rollator.

● DO

- (1) Always ensure the brakes are in the locked position if the rollator is left unattended.
- (2) Check all castors and wheels are secure before use.
- (3) Check the brakes are functioning properly.
- (4) Check handle height adjustment hand knobs are secure before use.
- (5) Check the rollator is fully unfolded.
- (6) Take care when carrying the rollator when folded.
- (7) Keep your hands away from the folding mechanism to prevent any risk of pinching.

• DO NOT

(1) Sit on the rollator if the brakes are not locked, this could cause the rollator to move resulting in injury to the user or other persons.

(2) Drop into the seat of the rollator, always sit into the rollator in a smooth, controlled motion. Seek assistance if necessary.

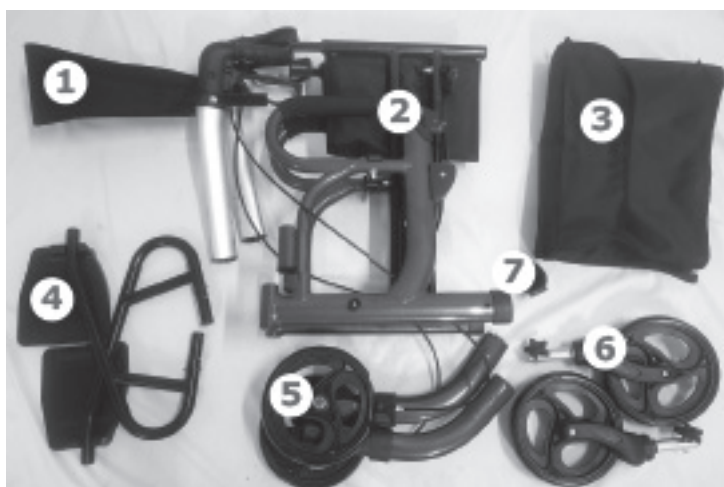
(3) Hanging objects on the brake levers or hand-grips, this could affect brake operation and/or stability of the rollator.

Every care has been taken to ensure that your rollator will provide safe and reliable service as you adhere to the advice given above.

• LIST OF COMPONENTS

1. two handle bars with backrest
2. main frame
3. shopping bag
4. a pair of footrest
5. a pair of rear legs
6. two front wheels
7. cane holder

Your rollator is equipped with various components, you should know these components and parts before proceeding to read this user manual. Design and specifications could be changed without further notices.


• SPECIFICATION

Hand grips inside width	19.7 in / 50 cm
Overall width	24.8 in / 63 cm
Overall depth	30.3 in / 77cm
Seat area	18.4*12.2 in / 46.7*31 cm
Seat height	22 in / 55.8 cm
Adjustable handle height	33.9-38.7 in / 86-98 cm
Max. user weight	352 lbs / 160 kg
Net weight	21 lbs / 9.5 kg

• ASSEMBLY INSTRUCTION

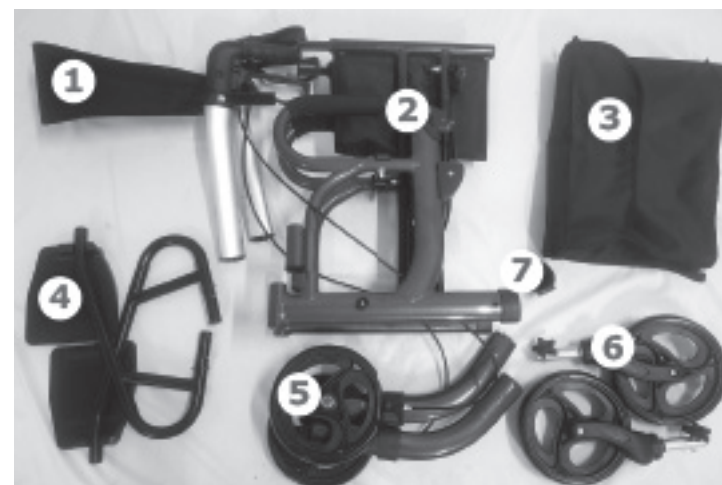1. Take out all packages.

2. Take out the main frame and press the sides of the seat, then the seat was secured at a click.

3. Press the red button at the bottom. Insert the handle bar into the main frame, adjust to lowest location.

4. Remove the rear tube, and fix the leg with the spring button, making sure the spring button is locked in the correct position.

5. Insert the front wheels into front tubes, then use star screw to fix them.

6. Install the crutches bracket on the left bracket.

7. Hang up the bag

8. Footplate assemble

Press the small button as picture showed, push the footplate down and hear a click and the footplate fits into place.

9. Remove Footplate

Use a pin insert the hole as shown, then pull up the footplate.

How to use footplate

Pull up the footplate, then rotate it to the correct position.

BACKREST ADJUSTABLE

Backrest can be 180 degree rotated.

HANDLE ADJUSTABLE

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set it to the desired height. Repeat for the opposite side and make sure that both handles are secure and set at the same height before use.

Tips:

When sitting and resting on the rollator, please adjust the height of the handrail to your favorite position so that you can sit on the chair in a more comfortable posture.

BRAKE OPERATION ADJUSTMENT

1. Pull up on brake handle lever to slow the rollator.
2. Push down on the brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock.
3. Brakes must be applied in the lock position when the seat is going to be used. Precheck the lock by pushing forward on the rollator.

Brake adjustment is carried out as follows:

Turn the adjusting screw anti-clockwise to increase the tension, then lock the adjustment locking nut clockwise whilst holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake position.

● MAINTENANCE

Over a period of time, the brake cables may stretch, and if no further adjustments can be made, the cables should be replaced. Seek the help of the manufacturer when replacing brake cables. The castors should be checked that they are secure each time before using the rollator. Also, check the brakes operate correctly and the handles and hand knobs are secure. Should any part of the rollator appear loose or insecure, discontinue use until it is inspected by a qualified technician. The following shows the maintenance schedule:

Weekly

- Check brake for malfunction
- Check all wheels and castors for loose bolts and nuts and knobs

Periodically

- Check hand grips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check tires for wear or cracks

Yearly

- Once a year have your rollator fully inspected, cleaned, maintained and lubricated with all moving parts

Serviceable Components

- Wheels
- Hand grips
- Bag
- Knobs
- Cane holder
- Standing armrest sleeve

CLEANING

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleaners may be used if required. Care should be taken not to leave any cleaning residues on the hand grips or seat (where fitted) which may leave them slippery when used. Do not use a scouring pad for scrapers this may damage surface coatings.

WARNINGS!

1. Maximum working limit 352lbs/160 kg.
2. Do NOT stand on this product.
3. Ensure both push handles are adjusted to the same height.
4. Seek advice from a trained healthcare professional for height adjustment of this product.
5. Always maintain a safe distance between your rollator and other pedestrians or objects.
6. Some components may cause harm or damage when they come into contact with other objects.
7. Do not attempt to replace the wheels or castors of your rollator. If they are faulty or require a replacement, please contact us.
8. Ensure that your rollator is opened fully (folding scissor arm is straight) before reusing it.
9. Ensure that your fingers are clear from the scissor arm when folding.

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EN

Return / Damage Claim Instructions

- ⚠ **DO NOT discard the box / original packaging.**
In case a return is required, the item must be returned in original box. Without this your return will not be accepted.
- ⚠ **Take a photo of the box markings.**
A photo of the markings (text) on the side of the box is required in case a part is needed for replacement. This helps our staff identify your product number to ensure you receive the correct parts.
- ⚠ **Take a photo of the damaged part (if applicable).**
A photo of the damage is always required to file a claim and get your replacement or refund processed quickly. Please make sure you have the box even if it is damaged.
- ⚠ **Send us an email with the images requested.**
Email us directly from marketplace where your item was purchased with the attached images and a description of your claim.

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