USA office: Fontana **AUS** office: Truganina **GBR** office: Ipswich **ITA** office: Milano **POL** office: Gdańsk
DEU office: FDS GmbH, Neuer Höltingbaum 36, 22143 Hamburg, Deutschland **FRA** office: Saint Vigor d'Ymonville

If you're having difficulty, our friendly
customer team is always here to help.

USA:cs.us@costway.com
AUS:cs.au@costway.com
GBR:cs.uk@costway.com
ITA:cs.it@costway.com

POL:sklep@costway.com
DEU:cs.de@costway.com
FRA:cs.fr@costway.com
ESP:cs.es@costway.com

Bed Safety Rail
Bettschutzgitter
Barrière de Sécurité pour Lit
Barandilla de Seguridad para Camas
Sponda di Sicurezza del Letto
Poręcz bezpieczeństwa do łóżka

JH10016WH

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION. PLEASE READ AND KEEP FOR FUTURE REFERENCE.

EN

Before You Start

- ⚠ Please read all instructions carefully.
- ⚠ Retain instructions for future reference.
- ⚠ Separate and count all parts and hardware.
- ⚠ Read through each step carefully and follow the proper order.
- ⚠ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⚠ Always place the product on a flat, steady and stable surface.
- ⚠ Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.

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EN

APPLICATION OF BED SAFETY RAIL

This Bed Safety Rail is an assistive balance device. It is designed to assist people who are mobile, but require some assistance to get in and get out of bed, or reposition in bed safely. The bed rail is NOT intended for people with very poor mobility, or who are permanently bedridden. It is NOT designed to prevent people from falling out of bed. Please keep this user instruction in a safe place for future reference.

-Material: carbon steel tube

-Max loads: 136kg (300 pounds)

CAUTIONS and WARNINGS

- Please make sure that you read and follow these instructions carefully. Failure to comply may result in injury.
- Ensure all parts are assembled securely and all spring buttons are properly engaged in position prior to use.
- The area between an assistive bed rail and mattress creates a potential for serious, life-threatening entrapment. It is possible for a person's head, neck, arm (s), leg (s) or body to become wedged between the side of the mattress and any of the bed rail.
- The mattress must be tight against the assistive handrail, with no gaps at all times. This should be checked frequently- at least daily.
- The bed rail is NOT designed to support the full weight of a person. Do not exceed the 136kg maximum weight capacity of the rail.

It is important that the Bed Safety Rail is not attached to any part of the bed that can move or be adjusted, and its attachment strap is not touching any part of the bed that moves.

PARTS LIST

ITEM	DESCRIPTION	QUANTITY
A	Bed Rail Handle	1
B	Side Handle	1
C	U-shape Frame	1
D	Base Frame	1
E	Extension Tube	2
F	Small C-Clip	2
G	Big C-Clip	2
H	Safety Strap	1

INSTALLATION

1. Take off two screws from side handle(B) firstly. Align holes in bed rail handle(A) with the holes in side handle(B), then secure two parts with previous screws.

Fig. 1


2. Insert two extension tubes (smaller ends) (E) into bed rail handle(A) and secure with two small C-clips(F).

Fig. 2

3.Insert U-shape frame(C) into base frame(D) and make sure that two spring buttons have fully engaged into the holes.

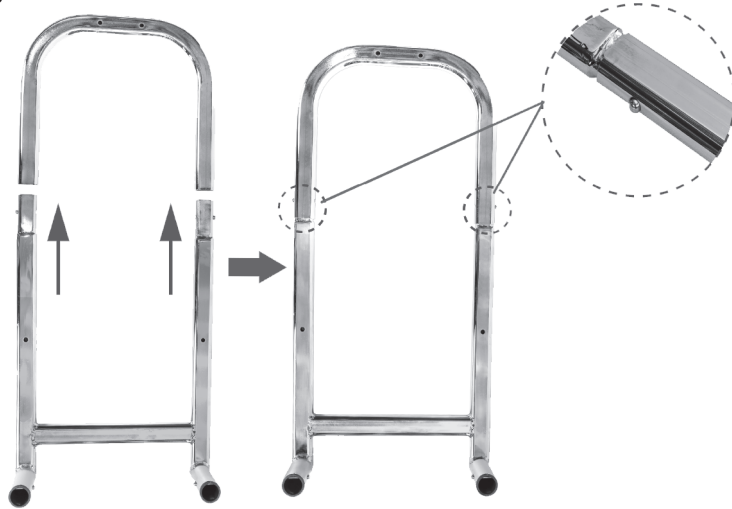Fig. 3

4.Slide bed rail handle(A) into both base frame's tubes(D) and secure with two big C-clips(G).

Fig. 4

5.Make sure all screws are fully tightened and all spring buttons are properly engaged in position prior to use.

Fig. 5

6.Remove the mattress and place the bed rail onto the base of the bed. Position the bed rail so the user can reach the handle from a relaxed lying position without having to stretch.

Fig. 6

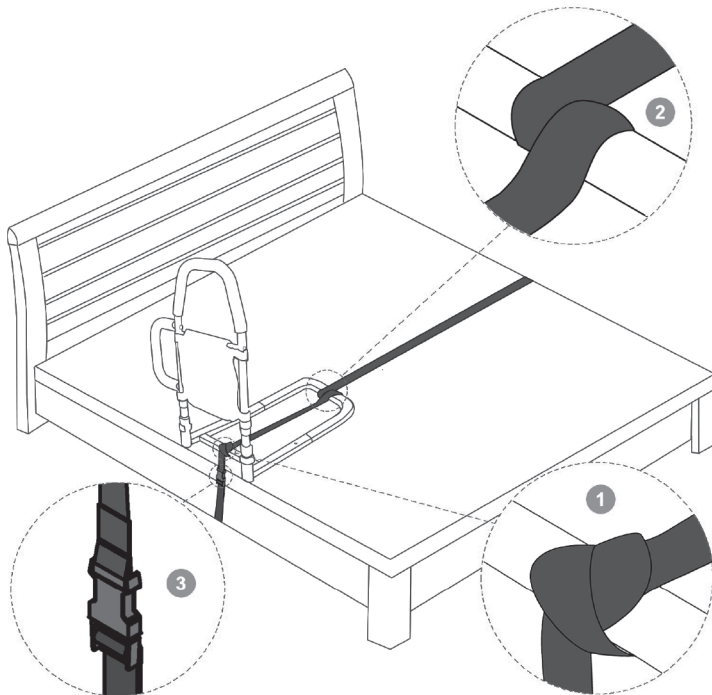
Option -Secure the Safety Strap to the Bed Frame (it's non-mandatory)

Take the mattress off the bed. Place the assembled bed safety rail at a desired location on your bed. Tie a secure knot on the the bottom rail in the position shown in step ①.

Loop the strap around the bed rail base in the position shown in step ②. Feed the strap over the top of the bed and back underneath the base. Buckle up the two strap buckles together as step ③. Pull down on the loose end of the strap to tighten.

Note: To avoid tripping hazards, make sure you tuck away any loose strap ends.

Option -Fix the Bed Rail on Bed Frame (it's non-mandatory)

The Bed Safety Rail could be fixed on base of the bed with screws provided and provide increased stability for users, but it is not necessary to do.

CLEANING THE BED SAFETY RAIL

Note: Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface.

- 1.Clean regularly using soapy water or warm water. Then wipe it with dry towel.
- 2.Allow product to fully dry prior to use/storage.

EN

Return / Damage Claim Instructions

- ⚠ **DO NOT discard the box / original packaging.**
In case a return is required, the item must be returned in original box. Without this your return will not be accepted.
- ⚠ **Take a photo of the box markings.**
A photo of the markings (text) on the side of the box is required in case a part is needed for replacement. This helps our staff identify your product number to ensure you receive the correct parts.
- ⚠ **Take a photo of the damaged part (if applicable).**
A photo of the damage is always required to file a claim and get your replacement or refund processed quickly. Please make sure you have the box even if it is damaged.
- ⚠ **Send us an email with the images requested.**
Email us directly from marketplace where your item was purchased with the attached images and a description of your claim.

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UK Responsible Person Information: Humiss Company Limited
MOBILITY SOURCE MEDICAL TECHNOLOGY CO., LTD