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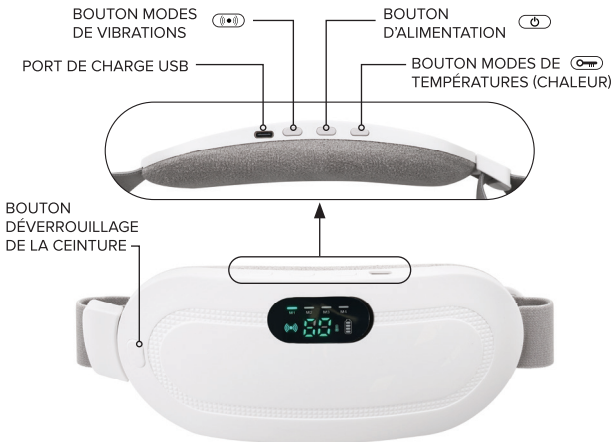
WEARABLE COMFORT.

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702658

**PORTABLE HEAT BELT
DURING YOUR MONTHLY CYCLE**

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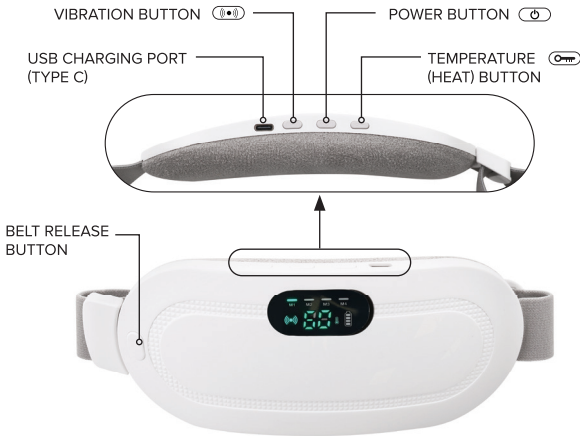
READ MANUAL CAREFULLY AND RETAIN FOR FUTURE REFERENCE

⚠ RISK OF SKIN BURNS: WHAT YOU NEED TO KNOW

Prolonged or improper use of heating massager belts can lead to thermal burns—even at relatively low temperatures. Discontinue use if heat causes any discomfort. Always turn off the unit if the heat becomes uncomfortable. Always start at the lowest setting. **DO NOT APPLY DIRECTLY TO BARE SKIN. DO NOT USE WHILE SLEEPING.**

SPECIFICATIONS

- Built-in rechargeable lithium 2500mAh battery
- Size: 185 x 85 x 25 mm (7.28" x 3.35" x 0.98")
- Includes USB charging cable
- Material: ABS, Velvet Microfibre

POWER ON/OFF

Press and hold down the power button to turn the unit ON or OFF. Unit is powered OFF when no LEDs are lit up on the readout.

TEMPERATURE SELECTION

- Press once – Temperature icon will light up – 60°C (140°F)
- Press a 2nd time – 55 °C (131°F)
- Press a 3rd time – 50°C (122°F)
- Press a 4th time – Turn heat function OFF

VIBRATION SELECTION

The vibration modes range from continuous gentle massage to pulsing vibrations for customizable relief.

- Press once – Vibration icon will light up – M1: Triple Pulse Mode
- Press a 2nd time – M2: Rapid Pulse Mode
- Press a 3rd time – M3: Continuous Mode
- Press a 4th time – M4: Slow Triple Pulse Mode
- Press a 5th time – Turn vibration function OFF

CHARGING

Plug in the USB Type C connection on the charging cable to the charging port on the unit. Connect the standard USB end into a 5W charging block or plug into a matching computer USB outlet.

Disconnect when charging is complete.

Battery indicator flashes red during charging. Indicator is solid red when charging is complete.

⚠ WARNINGS

- Keep out of reach of children and pets.
- Consult your healthcare provider before use if you are pregnant, suspect you may be pregnant, have sensory impairments, or specific skin conditions.
- Avoid prolonged high heat to prevent skin irritation or burns.
- Not suitable for people with certain conditions (e.g., sensory impairments or specific skin conditions) without medical advice.
- Avoid use on broken, irritated, or inflamed skin.
- Do not charge with a high speed charger. Use only a standard 5W charging connection.
- Do not wrap the cord around the product.
- Keep this product away from exposure to heat or moisture.
- Do not immerse in water or other liquid.
- Do not attempt repair. This unit contains no serviceable parts.
- Disconnect power when charging is complete or before cleaning the unit with a dry cloth.
- Do not apply excessive pressure.

1 YEAR WARRANTY

Relaxus Products Ltd. warrants this product to be free from manufacturer's defect for one year from the date of purchase. Original purchase receipt is required for any warranty claim.

Warranty excludes:

1. Damage caused by improper or incorrect installation or use.
2. Breakage or damage from mishandling or accidents.
3. Use of product for purposes for which it was not intended, or commercial use.

Relaxus Products Ltd. retains discretion to repair, replace parts or supply a new product of the same or similar model. Warranty is not transferable. Any associated shipping costs are not included in warranty coverage.

Distributed by Relaxus Products Ltd
1590 Powell Street,
Vancouver, BC, V5L1H3
Canada

Made in China / Fabriqué en Chine