


MIRACLE™ DRY-EYE RELIEF

Dear Customer,

Thank you for your purchase. We're here to help! If you have any problems or questions, please do not return to the store. Instead, contact us directly at: MiracleRelief@rephelpdesk.com

Thank you,
Miracle™ Dry-Eye Relief

WHATS INCLUDED:

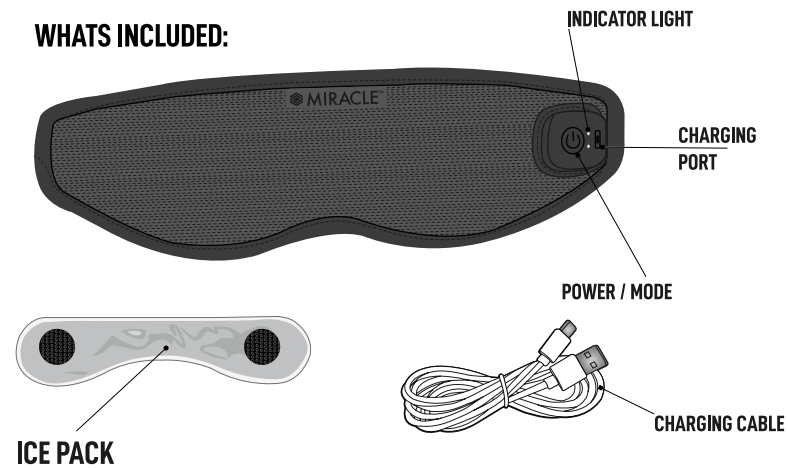

Charging


1. Insert small end of Charging Cable into Charging Port and large end into a USB port or USB power adapter (not included).
2. Indicator Light will blink while charging, and turn solid when fully charged.
3. When battery is low, indicator light will blink to let you know it is time to charge.

Setup

If planning to use the included ice pack, place in the freezer for approximately one hour before use to be sure it is frozen.


Use

1. Fully charge battery before use, following instructions in Charging section.
2. If using the included ice pack, place frozen ice pack with hook and loop side facing toward the mask, and press down to secure. **See Fig 1.**
3. Using the adjustable strap, wrap around your head and secure, adjusting length until the mask fits snug over your eyes, and using the hook and loop ends. **See Fig 2 and Fig 3.**
4. Press the power button to turn the mask on:

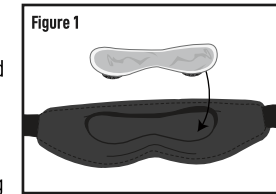

ADJUSTABLE STRAP

- 1st Press: Massage and Heat On (not recommended when using ice pack)**
2nd Press: Massage Only On
3rd Press: Heat Only On (not recommended when using ice pack)
4th Press: Off

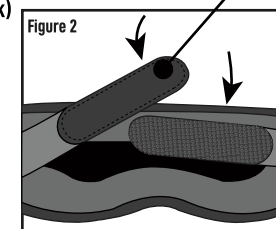

Care


Wipe with damp cloth and allow to air dry. If you do not plan to use for an extended period of time, store in a cool, dark place.

Not machine washable. Do not Dry Clean.

Warning :

Failure to follow instructions may cause overheating fire or personal injury.

- Do not use if wet.
- Do not trap or pinch cord.
- Do not iron.
- Do not bleach.
- Do not use this with an infant, a helpless person, or anyone insensitive to heat, such as a person with poor blood circulation. If improper operation is observed, discontinue its use immediately.
- Do not dry clean, cleaning solvents may have a deteriorating effect on the insulation of the heating element.

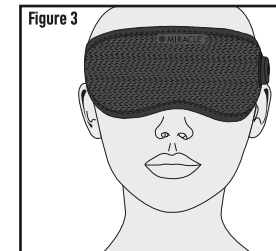

Distributed by
Onitel
PRODUCTS CORPORATION

©2024 Onitel Products Corp.
Fairfield, NJ 07004
All Rights Reserved.
Made In China
MEYR-IM-01-0824-400