

MSA Hydro™ Ear Wax Remover

By Ear Basics®

Thank you for choosing our MSA Hydro™ Ear Wax Remover. Our Innovative ear-cleaning device uses a gentle 5-Stream water nozzle to remove ear wax effectively, easily, and safely.

This instructional manual will guide you through the setup, proper use, and maintenance of your MSA Hydro™ Ear Wax Remover by Ear Basics®.

Included:


- 1 - MSA Hydro™ Ear Wax Remover Wand
- 2 - MSA Hydro™ Ear Wax Remover Water Nozzles
- 5 - MSA Hydro™ Ear Wax Remover 5-Stream Tips
- 1 - MSA Water Shield
- 1 - MSA Charging Cord

Important Safety Information:

- Read all instructions carefully before using this device to avoid injury or damage caused by improper operation.
- **DO NOT USE** the MSA Hydro™ Ear Wax Remover if you have any ear infections, injuries, or an ear condition without consulting a healthcare professional.
- Keep out of reach of children.
- Avoid direct contact of the water stream with your eardrum.
- **Do Not** submerge the device in water.
- Use only clean, lukewarm water for ear wax removal.
- **Disconnect** the device from the charger before cleaning or maintenance.
- **DO NOT** disassemble the product to avoid product damage or electrical shock.
- **DO NOT** place the product near fire, as the battery can be damaged or explode.
- **DO NOT** use the product for any other purpose than ear wax removal.
- **DO NOT** use if the power cord is damaged.
- **NEVER** use this product without the 5-Stream nozzle attached to the nozzle wand.
- **DO NOT** bend, stretch, or twist the nozzle wand.
- **ONLY** use warm water. Room Temperature or Cold Water may cause dizziness.

Product Overview:

- Water Reservoir
- Nozzle Wand
- 5-Stream Water Tip
- Water Shield
- Control Panel
- On/Off Button
- Water Pressure Control Button
- Charging Port
- Indicator Lights
 - Charging Indicator
 - Battery Status Indicator
- Hand Grip

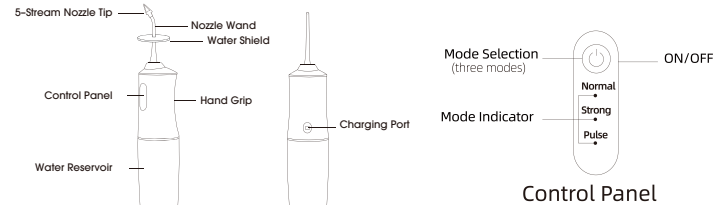

Charging Instructions:

1. Initial Charge: Charge the MSA Hydro™ Ear Wax Remover for 4 hours before first use.
2. Charging:
 - a. Connect the USB charging cable to the charging port on the wand
 - b. Plug the USB end into a power source.
 - c. The charging indicator light will illuminate.
3. Full Charge will be indicated by the three lights illuminating.
4. Regular Use: A full charge should provide up to 50 minutes of continuous use with a full water reservoir.

How to Use the MSA Hydro™ Ear Wax Remover

Step 1: Fill the Water Reservoir

- Remove the water reservoir by twisting/rotating the water reservoir clockwise
- Fill the water reservoir with clean, lukewarm water
- Replace the water reservoir and twist it counterclockwise to lock it securely

Step 2: Insert the Nozzle Wand Into the Handle

- Ensure that the nozzle wand is inserted firmly to avoid water leaking during use

Step 3: Place the MSA Water Shield on the nozzle wand as seen in the diagram above

Step 4: Attach 5-Stream Nozzle Tip

- Firmly place the 5-Stream nozzle tip onto the end of the Nozzle Wand

Prepare the Device

- Press the ON/OFF button to turn on the MSA Hydro™ Ear Wax Remover. The device will turn on and be in the normal position and the indicator light will illuminate.
- To switch to another mode, hold the ON/OFF button down for 2 seconds, and the mode will change from Normal to Strong. Press the ON/OFF button again for 2 seconds, and the mode will change from Strong to Pulse.
- Device will turn off automatically after 2 minutes.

Cleaning Your Ears

- Tilt your head slightly to one side
- Gently insert the 5-Stream water nozzle tip into your ear canal
- Press the ON/OFF button to activate the water stream
- Move the nozzle tip in small circles to thoroughly clean the ear canal
- Repeat on the other ear

Drying your Ears

- Use a clean dry towel to wipe the outer ear after use
- Allow the inner ear to air dry naturally

Cleaning and Maintenance

- After each use empty the water reservoir and rinse with clean water. Wipe the device with a wet cloth and dry thoroughly.
- Each week clean the water nozzle and 5-Stream nozzle tip with antibacterial solution.
- Every 90 days the 5-Stream nozzle tip should be replaced.
- If the device is not used for a long period of time, ensure that the water reservoir is dry and waterless so that when stored there is no bacteria growth.
- Store with an empty, dry water reservoir and place in a cool dry place.

Troubleshooting

- If the device does not turn on ensure that the device is fully charged, which can take up to 4 hours. Also check the charging port and cable for any damage.
- If the water pressure seems to be weak, check that the water reservoir is filled and then adjust the water pressure control to a different mode. Also check to ensure that there is no blockage in the nozzle wand or 5-Stream nozzle tip.
- If you detect any water leaking, ensure that the water reservoir is securely closed and locked, that the nozzle wand is inserted correctly, and the 5-Stream nozzle tip is secure.

Thank you for choosing the MSA Hydro™ Ear Wax Remover. We are committed to making life simpler through innovation.

 Warning: Cancer and Reproductive Harm – www.P65Warnings.ca.gov.
Due to Continuous Product Improvement, the Appearance of the Actual Product May Differ Slightly.

Designed in USA Made Responsibly in China
Global Concepts Limited, Inc. Coconut Creek, FL 33073
© 2023 Global Concepts Limited, Inc. All Rights Reserved
All photographs & graphics are the property of their respective owner infringer's will be prosecuted

0524-3118